


# Loading Guide


for Multiject Reusable Injector with VISCOJECT 2.2 Disposable Cartridge and enVista® (MX60)

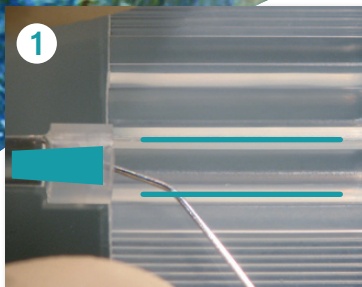


1 Apply two thin lines of Amvisc® Plus or other Bausch + Lomb viscoelastic into the lateral grooves within the loading chamber. Then, apply a small amount of the viscoelastic solution directly to the opening of the cartridge tunnel.




2 Open the vial containing the IOL and, using non-serrated forceps, remove the lens by grasping and carefully pulling it out vertically from the center slot at the top portion of the vial.

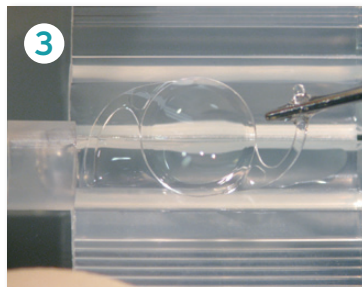

3 Position the lens in the middle of the loading chamber so that the anterior side is up and the lens is in a reverse "s" orientation.

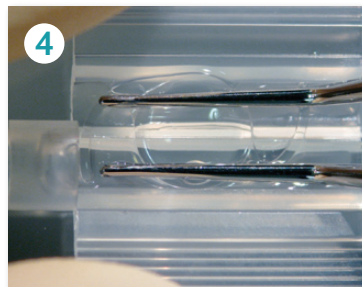


4 Apply slight downward pressure with the forceps to push the lens and haptics down and ensure they are properly seated under the grooves.

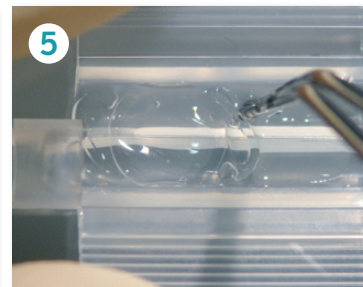


5 Using a non-serrated forceps, move the lens toward the nozzle such that the leading and trailing haptics are compressed. Both haptics should be pointing toward the optic.


Ensure that the lens is correctly aligned and that the cartridge sides close. Make sure that neither the edge of the lens nor the haptic are wedged in.

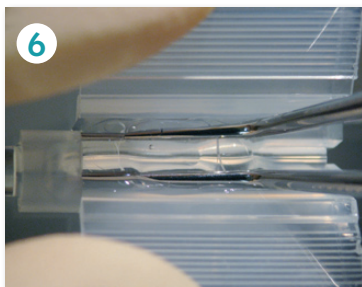


6 Next, close the cartridge wings together until the "click-lock" mechanism engages.

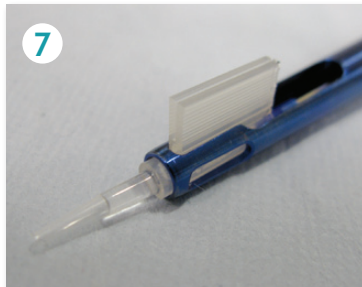


7 Insert the cartridge into the front end of the injector. Push the cartridge into the very front position.

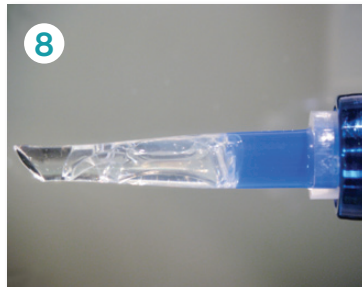

8 Slide the plunger forward and ensure that the silicone cushion correctly enters the loading chamber. Rotate the screw until the screw thread catches

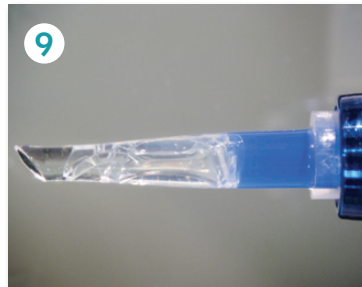

9 Now, pull the plunger back sufficiently to visually confirm that the lens remains in the conical tip and then push forward again. This step also ensures that the trailing haptic is not trapped under the plunger tip. The lens is now loaded and ready for injection.

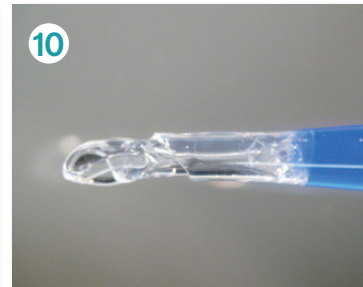

10 With the conical tip bevel facing down, inject the lens by clockwise rotation of the screw mechanism until the lens is fully expressed from the tip.

Carefully remove the injector tip from the eye.

## Precautions

**VISCOJECT Cartridge:** Do not reuse or resterilize the disposable cartridge. Single use only. Refer to the manufacturer's Directions for Use for detailed instructions.

**Multiject Handpiece:** Follow the manufacturer's recommended sterilization, inspection, handling and storage techniques for the reusable handpiece as outlined in the Directions for Use. Refer to the manufacturer's Directions for Use for detailed instructions.

©2012 Bausch & Lomb Incorporated. ®/™ are trademarks of Bausch & Lomb Incorporated or its affiliates. All other products/brands are trademarks of their respective owners. SU6829 12/12